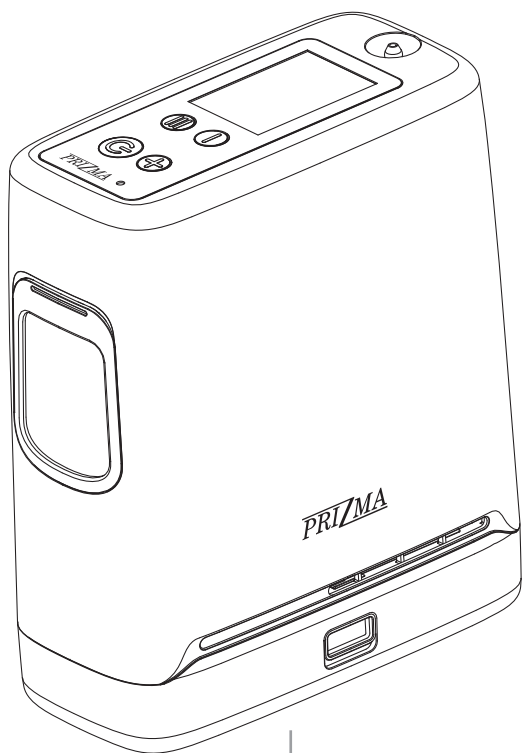
PRI/MA O₂

PORTABLE
OXYGEN
CONCENTRATOR

PRENOSIVI
KONCENTRATOR
KISEONIKA

Instruction manual
Uputstvo za upotrebu

CE 1304

Please read this manual carefully before using the product.

The manufacturer reserves the right to interpret and revise these instructions.

Please always keep this manual with the product.

The manufacturer may update the product's technology or software without prior notice.

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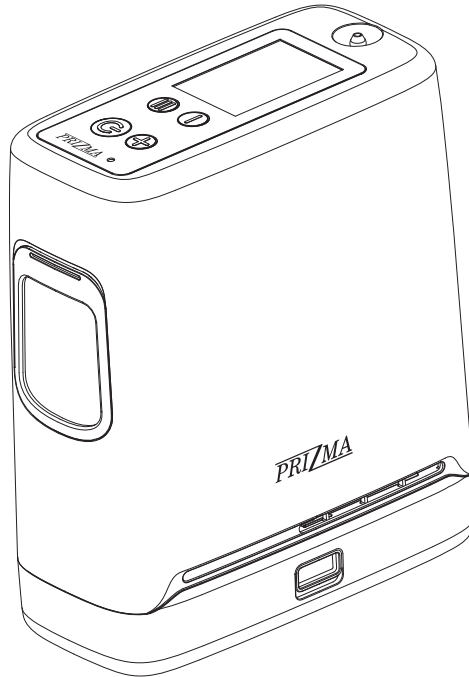
Foreword

Thank you for purchasing our product.

This manual contains a description of the product, precautions, operation steps, specifications, as well as basic maintenance and troubleshooting instructions.

To ensure safe and effective use of the device, please read and understand this manual before using the product. Please pay special attention to the “Safety Instructions” chapter.

If the product needs to be returned for repair, please refer to the chapter “Other servicing and maintenance activities” and return the product according to the instructions provided.

If you have any questions about the use of the product, please contact the manufacturer, representative or distributor.

Important

Users should read this entire manual before operating the PRIZMA O2 Portable Oxygen Concentrator. Failure to do so could result in personal injury. If you have questions about the information in this instruction manual or about the safe operation of this system, contact your equipment provider.

Make sure you have a backup oxygen supply in addition to this portable oxygen concentrator.

Contents





1. GLOSSARY OF SYMBOLS	3	7. CLEANING AND MAINTENANCE	22
2. QUICK START-UP	4	7.1 Oxygen tube and nasal cannula	22
3. SAFETY INSTRUCTIONS	5	7.2 Housing of the oxygen concentrator	22
4. DESCRIPTION	8	7.3 Inlet air meshes	23
4.1 Intended use	8	7.4 Inlet air filter	23
4.2 Exclusions and contraindications	8	7.5 Output filter	24
4.3 Principle of operation	9	7.6 Replacement of the molecular sieve module	24
5. STRUCTURE OF PRIZMA O2 POC	10	7.6.1 Disassembly of the molecular sieve module	24
5.1 User interface	10	7.6.2 Assembly of the molecular sieve module	25
5.1.1 Display	10	7.7 Other servicing and maintenance activities	26
5.1.2 On/Off button	11	8. QUICK TROUBLESHOOTING	27
5.1.3 Function button	11	9. EXPLANATION OF NOTIFICATIONS AND ALARMS	28
5.1.4 Oxygen flow setting control buttons	11	9.1 Notifications	28
5.1.5 Light indications	11	9.2 Low priority alarms	29
5.1.6 Audio indications	12	9.3 Medium priority alarms	30
5.2 Input/output connections	12	9.4 Informational signals	32
5.2.1 Air inlet mesh	12	10. ENVIRONMENT PROTECTION	32
5.2.2 Oxygen tube connector	12	11. ELECTROMAGNETIC COMPATIBILITY	33
5.2.3 DC supply plug	12	12. SPECIFICATIONS	37
5.2.4 Battery	12		
5.2.5 AC/DC adapter	13		
5.2.6 Oxygen tube and nasal cannula	14		
6. OPERATING INSTRUCTIONS	15		
6.1 General instructions	15		
6.2 Battery installation/removal	15		
6.3 Connection of external DC power supply	16		
6.4 Connection/disconnection of oxygen tube to/from the oxygen concentrator	17		
6.5 Turning the oxygen concentrator on	17		
6.6 Setting of oxygen flow rate	18		
6.7 Application of nasal cannula on your face and breathing through nose	18		
6.8 Turning off the oxygen concentrator	19		
6.9 Guidelines for use, maintenance and storage of batteries	20		

1. Glossary of Symbols

Symbol	Meaning	Symbol	Meaning
 WARNING	Indicates hazard or unsafe practice that can result in severe body injury or death.	 CAUTION	Indicates hazard or unsafe practice that can result in minor body injury or property damage.
	Refer to instruction manual / booklet		AC Power
	Keep dry		DC Power
			Maximum and minimum temperature limits at which the item shall be stored, transported or used
IP22	Protected from touch by fingers and objects greater than 12.5 mm (0.5"). Protected from dripping water less than 15 degrees from vertical.		
	Atmospheric pressure limitation to which the medical device can be safely exposed (operating).		Relative air humidity limitation to which the medical device can be safely exposed (operating).
	No Smoking.		No oil or grease.
	No open flame (oxygen concentrator). Do not incinerate (battery).		Do not disassemble.
	Manufacturer		European conformity
	Type BF applied part		Class II equipment
	Packaging is recyclable.		Unique Device Identification (01)08606111792555(11)YYMMDD(21)NNNNNN (01) GTIN - Global Trade Item Number (11) Manufacture date, YYMMDD (21) Serial number, NNNNNN
	Do not dispose of in unsorted municipal waste. Waste Electrical and Electronic Equipment (WEEE)		This side up
	Medical device		Authorized Representative in the European Community

For icons displayed on the user interface panel refer to the section 9.

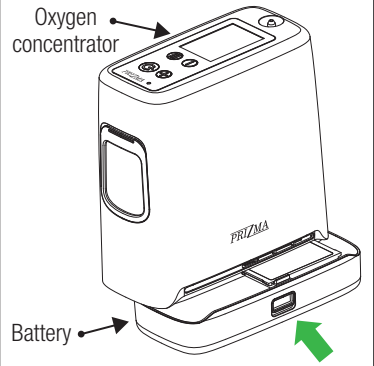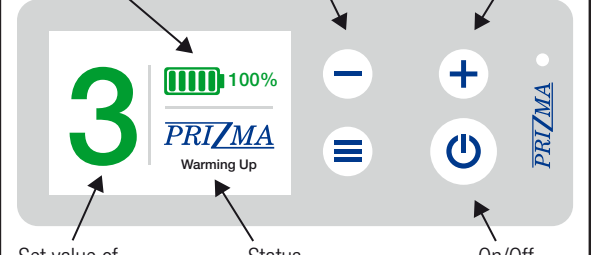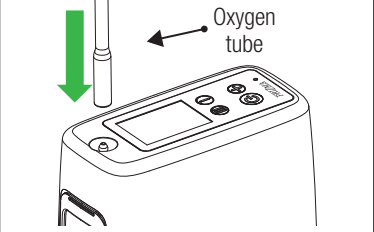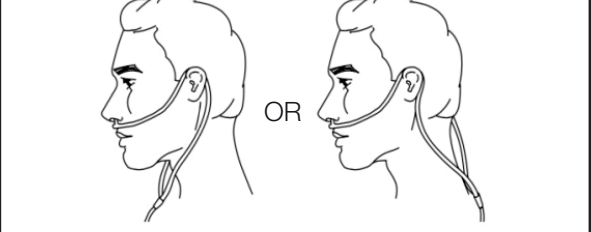2. Quick Start-Up

<p>1</p> <p>Install the charged battery into the slot at the bottom of the oxygen concentrator case. Slide the battery in until it is latched into the oxygen concentrator (a clicking sound will be heard).</p> 	<p>2</p> <p>Press and hold the On/Off button for a few seconds until the oxygen concentrator turns on. Set the desired oxygen flow value (levels 1–6) using the - and + buttons. The oxygen concentrator requires a warm-up period. The warm-up status is displayed under the manufacturer's logo.</p> 
<p>3</p> <p>Connect the oxygen tube to the oxygen concentrator.</p> 	<p>4</p> <p>Insert the nasal cannula into the nostrils and start breathing through the nose.</p> 
<p>5</p> <p>After the concentrator has warmed up, the status under the manufacturer's logo will change to "Normal Running," and oxygen delivery will begin.</p>	

3. Safety Instructions

**WARNING**

Please keep this manual with the product at all times. Do not use the product without first reading the user manual. Do not operate this device if you are unsure of its operation or function. Contact the device manufacturer or retailer for assistance or further information.

**WARNING**

The User Manual for this device has been written taking into account the training and knowledge level of the intended patient population to ensure proper device operation. No additional user training is required for device use beyond following the guidelines provided in the user manual.

**WARNING**

This product is not a life-support device. It is recommended to always follow doctor's advice for setting of oxygen flow for each patient individually.

**WARNING**

Patient with long term needs for oxygen should have backup oxygen supply (oxygen cylinders, oxygen bags, etc.).

**WARNING**

The settings of other models or brands of oxygen therapy equipment may not correspond to the settings of this device.

**WARNING**

The settings of this device may not correspond to the settings for devices that provide continuous-flow oxygen.

**WARNING**

The efficiency of oxygen treatment should be monitored by an external pulse oximeter. In case of any adverse reaction, IMMEDIATELY inform a doctor.

**WARNING**

If you experience discomfort or observe equipment not working properly, IMMEDIATELY stop using this product and contact the equipment supplier or doctor.

**WARNING**

Use only voltages specified on the rating label. Use only supplied AC/DC adapter.

**WARNING**

Do not use this product if power cords or plugs are damaged. Do not put heavy objects over power cords. Improper use of power cords and plugs may cause fire, electric shocks, burns and other hazards.

**WARNING**

Do not remove covers while the product is plugged in. There are no user-serviceable parts under protective covers. Servicing should be done only by qualified personnel. PRIZMA will provide circuit diagrams, component part lists, descriptions and instructions to assist service personnel in parts repair.

**WARNING**

PRIZMA O2 is MR unsafe. Do not expose to MRI equipment or other devices that generate strong magnetic fields (for example, X-ray, CT scan, or other types of radiation).

**WARNING**

Before cleaning or servicing, unplug the oxygen concentrator.

**WARNING**

Take care to prevent the product from getting wet, and do not allow water to enter into it. Do not install or store the oxygen generator in a place where water or other liquids can drip into it.

**WARNING**

Avoid using the oxygen concentrator when bathing. However, if you need to use it, the oxygen concentrator and AC/DC adapter must be located outside the bathroom at a minimum distance of 2.5 meters from the bathtub or shower.

**WARNING**

Do not leave the oxygen concentrator unattended after it is powered on.

**WARNING**

This product generates a high concentration of oxygen, which supports combustion. Keep the operating oxygen concentrator away from open flames and sources of fire. If there are high temperature objects, sparks, or open flames within a range of 2 meters from the oxygen concentrator, DO NOT use this device.

**WARNING**

Smoking during oxygen therapy is dangerous and is likely to result in facial burns or death. Do not allow smoking within the same room where the oxygen concentrator or any oxygen-carrying accessories are located. If you are unable to leave the room, you must wait 10 minutes after turning the oxygen concentrator off.

**WARNING**

Do not leave the oxygen tube under the bed cover or chair cushion. If the oxygen concentrator is turned on when not in use, a high oxygen concentration can build up and cause flammable materials to catch fire. When no one is inhaling oxygen, it is recommended to turn off the oxygen concentrator and disconnect the AC/DC adapter.

**WARNING**

Use only spare parts recommended by the manufacturer to ensure proper function and to avoid the risk of fire and burns.

**WARNING**

Do not use lubricants, grease, petroleum-based products or flammable gases on, or near the oxygen concentrator.

**WARNING**

Use only water-based lotions or salves that are oxygen-compatible before and during oxygen therapy. Never use petroleum or oil-based lotions or salves to avoid the risk of fire and burns.

**WARNING**

Ensure that the oxygen tube and nasal cannula are firmly connected to the oxygen concentrator and are not obstructed. If the oxygen tube is under covers, cushions, pressed by some heavy object, collapsed or obstructed, there is a risk of obstruction of delivery of oxygen to the patient.

**WARNING**

Do not share the same oxygen tube and nasal cannula with other patients. There is risk of cross-infection by viruses or bacteria between patients.

**WARNING**

In case of nosebleeding consult your physician.

**WARNING**

The responsible organization is accountable for ensuring the compatibility of the oxygen concentrator and all of the parts or accessories used to connect to the patient before use.

**WARNING**

Exposure to lint and dust may impair the device's performance. Avoid using or storing the device in dusty or lint-prone environments, and keep it in a clean, dry location when not in use.

**WARNING**

Direct exposure to sunlight or intense artificial light may reduce the device's lifespan. Store the device away from direct light, especially sunlight, when not in use.

**WARNING**

Altitude above 3000 m, temperature outside the range of 5 – 40 °C, or a relative humidity above 95%, will adversely affect the flowrate and the percentage of oxygen and the quality of the oxygen therapy. Use of this device immediately after storage in temperatures outside the allowable operating range may adversely affect operation of the device until the temperature returns to the allowable operating range. Wind or strong drafts can adversely affect the accuracy of delivery of oxygen.

**CAUTION**

Do not place the oxygen concentrator in environment with obstructed airflow.

**CAUTION**

Always place the oxygen concentrator on solid surfaces. Never place the oxygen concentrator on surfaces like beds or cushions as there is a risk that it might tip over or fall.

**CAUTION**

Ensure that power cords do not obstruct walkways or the movement of other people. It is recommended that the AC/DC adapter and the oxygen concentrator should be placed near the power outlet which is closest to the user.

**CAUTION**

The oxygen concentrator contains electronic components and may be affected by radio frequency interference (RFI). Always exercise caution when using the oxygen concentrator in the vicinity of portable communication equipment.

4. Description

4.1 Intended use

The intended use of the PRIZMA O2 Portable Oxygen Concentrator is to provide a supplemental supply of high-concentration oxygen to patients with indicated hypoxemia, through a nasal cannula, based on a physician's report in accordance with medical prescription and instructions.

The device is indicated for patients with chronic or acute hypoxemia resulting from pulmonary or cardiac disorders.

The device is intended for personal use.

The intended patient is an adult, spontaneously breathing (able to inhale and exhale without the use of machine), conscious person requiring supplemental oxygen therapy.

The intended user is an adult individual who is able to read and understand the instructions for use and to operate the device according to the instructions and parameters prescribed by a physician.

The device does not require installation by the manufacturer or specific user training, other than reading and understanding the instructions for use and operating the device in accordance with the physician's instructions.

The device is small and portable and may be used at home, in institutions and in various environments suitable for use of portable device. The device can be powered in three ways: by its battery, via the AC mains supply, or in a vehicle through a cigarette lighter socket (12V auxiliary DC power outlet).

The device is intended for use by a single patient but may be used by multiple patients when appropriately cleaned. The device should be cleaned between patients in accordance with the instructions provided in the instruction manual.

4.2 Exclusions and contraindications

- The device must not be used for patients who are unable to breathe spontaneously or are unconscious.
- The device must not be used for patients suffering from severe carbon monoxide poisoning.
- The device must not be used for patients with hypercapnia requiring controlled oxygen therapy.
- The device must not be used for patients with a tracheostomy.
- The device must not be used for patients who are susceptible to oxygen toxicity.
- The device must not be used for individuals whose resting breathing cannot trigger oxygen delivery.

- The device must not be used in combination with flammable anesthetics or other flammable gases or materials.
- The device is not intended for life-sustaining or life-supporting use.
- The device is not intended for oxygen therapy requiring more than 5 L/min continuous-flow.
- The device must not be used in combination with a humidifier, nebulizer, CPAP, other oxygen concentrators, or oxygen therapy devices. Such use may interfere with device operation or cause damage to the device.

4.3 Principle of operation


The operation of the oxygen concentrator is based on a molecular gas sieve for extracting oxygen from the ambient air using a pressure swing adsorption (PSA) process.

The molecular sieve is contained inside a sealed container. During the increase of air pressure in the sealed container, the molecular sieve adsorbs nitrogen from the air, while oxygen from the air stays in gaseous form. After the desired concentration level has been reached, oxygen at a concentration of 87-96% is ready to be delivered to the patient. The device detects start of patient's breath intake and at that moment delivers controlled and adjustable pulsed volume of oxygen from the sealed container to the patient. The collected nitrogen saturates the molecular sieve, which must then be regenerated. This is achieved by exposing the sealed container to a vacuum. During this phase, the collected nitrogen is released from the molecular sieve and expelled into the surrounding atmosphere. These steps of molecular sieve operation are repeated cyclically.


If the device is switched ON and no breathing is detected, it will enter Auto Breath Mode and dispense oxygen at regular intervals until breathing is detected again or the battery is depleted.

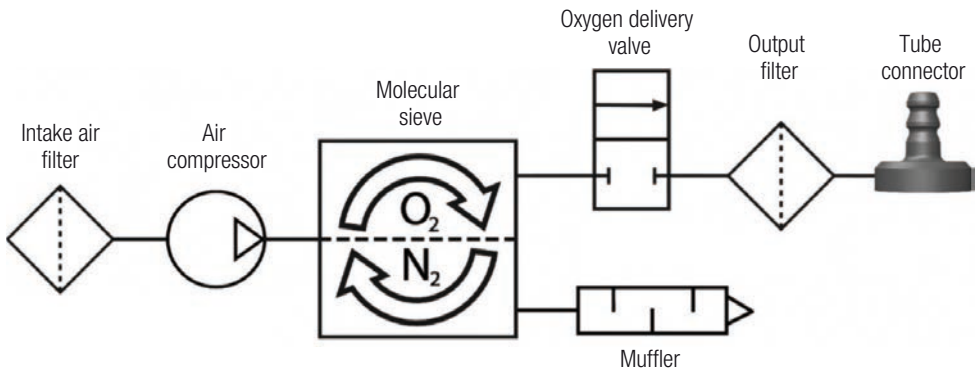Picture 4.3.1 Pneumatic diagram of oxygen concentrator

5. Structure of PRIZMA O2 POC

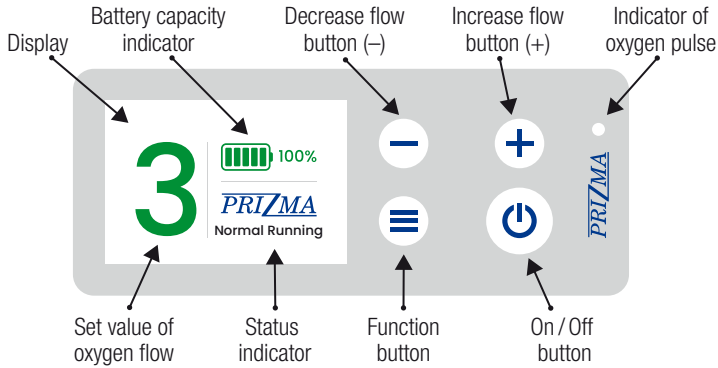
5.1 User interface

Picture 5.1.1 Commands and the home screen

5.1.1 Display

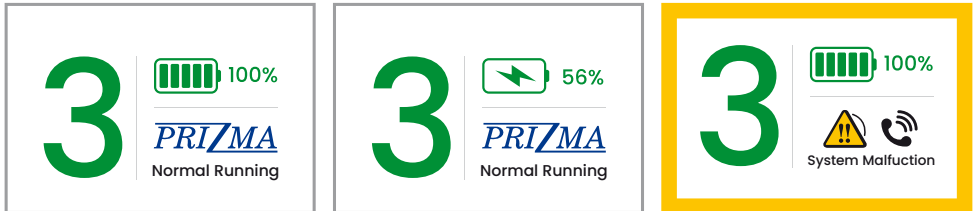This LCD screen displays information about the flow setting, battery capacity, battery charging status, operating status, and alarm and error notifications.

The set oxygen delivery level is displayed on the left side of the home screen.

The battery status is shown in the upper right part of the screen and indicates the current battery capacity. When the oxygen concentrator operates on battery power, a bar graph corresponding to the percentage of remaining battery capacity is displayed inside the battery symbol.

Appearance of battery status indicator	Remaining battery capacity	Appearance of battery status indicator	Remaining battery capacity
5 green bars	95 - 100 %	1 green bar	20 - 40 %
4 green bars	80 - 95 %	1 red bar	10 - 20 %
3 green bars	60 - 80 %	1 red bar, audible alarm	5 - 10 %
2 green bars	40 - 60 %	1 red bar, audible alarm, "Low Battery Shutdown" message	0 - 5 %

If the external DC power supply is connected to the oxygen concentrator, a lightning symbol will appear inside the battery icon. To the right of the battery symbol, a numerical value indicating the percentage of remaining battery capacity is displayed.

The status of the oxygen concentrator is in the lower right section of screen. If the device is operating normally, this section displays the manufacturer's logo. In case of an operating problem, the screen will show an alarm indication and a colored border around the screen. A flashing orange border indicates high-priority alarms.

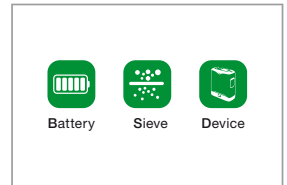
NOTE: During operation, the device will automatically turn off the display after a period of time to save power. To view the battery status, briefly press the On/Off button .

5.1.2 On/Off button

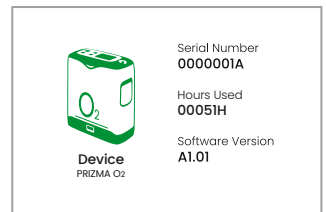
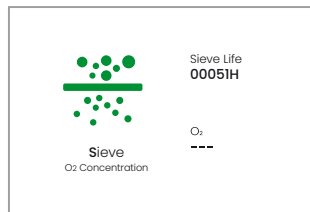
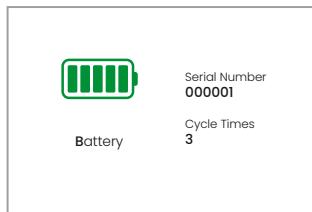
To turn the device on, press and hold the On/Off button  until the oxygen concentrator starts operating. To turn the device off, press and hold the On/Off button  until the oxygen concentrator starts shutting down.

5.1.3 Function button

Press once  to enter the menu for selecting information screens for the battery, molecular sieve, and the device.

Press Increase flow  button to select the desired information and press the Function button  to confirm. The selected information screen will then appear on the display.

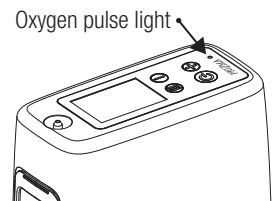
Press the Decrease Flow button  to return to the main screen.

5.1.4 Oxygen flow setting control buttons

Use  and  flow setting control buttons to set the oxygen flow level (1-6). The selected value is displayed as a large digit on the left side of the home screen.

5.1.5 Light indications

A green light at the top of the oxygen concentrator indicates the generation of an oxygen pulse.


5.1.6 Audio indications

An audible signal (beep) indicates a change in operating status or an alarm (a condition that may require user response). The frequency of the beeps depends on the priority level of the alarm. Higher-priority alarms are indicated by a higher beep frequency.

5.2 Input/output connections


5.2.1 Air inlet mesh

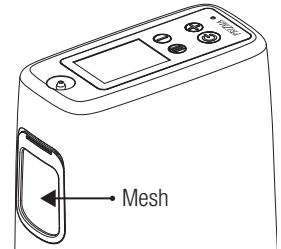There are two inlet air meshes on the sides of the oxygen concentrator. To keep the intake air clean of coarse particles, the inlet air meshes must be installed during operation of the oxygen concentrator.


5.2.2 Oxygen tube connector

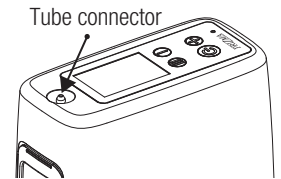The oxygen tube connector is located on top of the oxygen concentrator. The oxygen tube should be firmly attached to the nipple of the oxygen tube connector.

5.2.3 DC supply plug

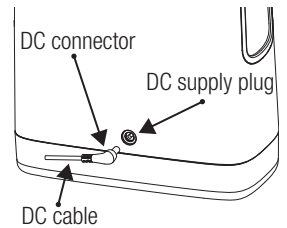The DC supply plug for connecting an external DC power supply is located on the back of the oxygen concentrator.

5.2.4 Battery

The oxygen concentrator can operate on battery power without connection to an external electric power supply.

The oxygen concentrator is shipped with a battery.

When new and fully charged, the battery provides up to 5 hours of operation at oxygen flow level 1.

The battery should be recharged only using the supplied external AC/DC adapter. The recharge time is up to 2.5 hours. A detailed description of battery charging is provided in the section "Guidelines for use, maintenance and storage of batteries".

5.2.5 AC/DC adapter

Use the PRIZMA O2 oxygen concentrator only with the supplied AC/DC adapter.

WARNING

WARNING

Do not use power supplies or cables other than those specified in this manual. The use of non-specified power supplies or power cables may create a safety hazard and/or impair device performance. Do not wrap cords around the power supply for storage. Do not drag or place objects on cords. Doing so may damage the cords and interrupt the power supply to the oxygen concentrator. To avoid risk of choking or strangulation, keep cords away from children and pets.

WARNING

Due to regulatory requirements, the AC/DC adapter cannot be used to charge the battery when the oxygen concentrator is used on an aircraft.

The AC/DC power supply provides the precise current and voltage required for safe operation of the oxygen concentrator when it is powered from the AC mains. The AC/DC adapter automatically adapts to mains voltages from 100 V to 240 V (50–60 Hz), permitting use worldwide.

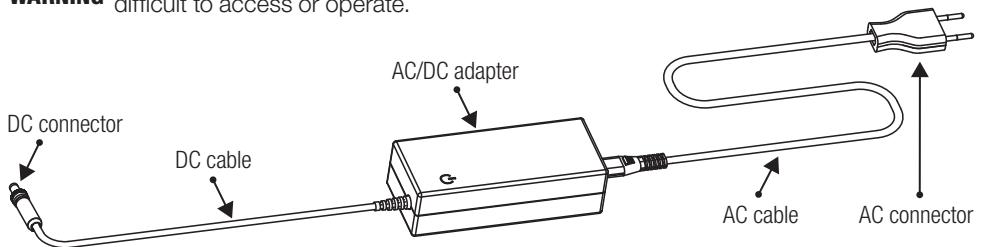
The AC/DC adapter also charges the battery when it is connected to the AC mains and the oxygen concentrator.

The AC input cable is detachable from the AC/DC adapter. The shape of the AC power connector depends on national standards.


WARNING

The mains plug of the power supply unit serves as the means of disconnection from the mains. Ensure that the socket outlet where the device is connected is easily accessible to the operator. Avoid positioning the mains plug in a way that makes it difficult to access or operate.

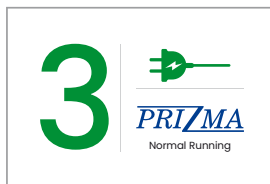NOTE: The device can operate using an external DC power supply only, without a battery. In this case, a plug icon will be displayed on the home screen.

5.2.6 Oxygen tube and nasal cannula

The oxygen tube and nasal cannula must be used with oxygen concentrator PRIZMA O2 to provide oxygen to the patient. It is recommended to use the supplied oxygen tube and nasal cannula to ensure proper breath detection and oxygen delivery. It is also possible to use other CE-approved nasal cannulas and oxygen tubes of similar length to the supplied ones.

It is important to regularly clean and disinfect the oxygen tube and nasal cannula to prevent infections.


**WARNING**

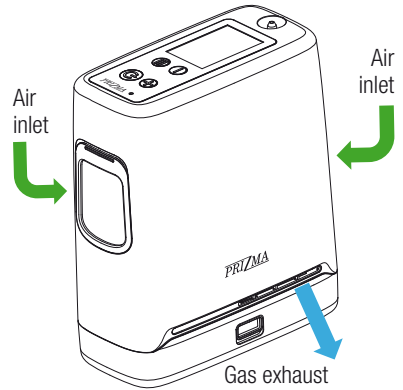If the oxygen concentrator is used by more than one patient, it is forbidden to use the same oxygen tube and nasal cannula by different patients. There is a risk of cross-contamination.

6. Operating Instructions

6.1 General instructions

- Place the oxygen concentrator in a well-ventilated location.
- Always operate the oxygen concentrator in an upright position.
- The air inlet and gas outlet must not be obstructed.
- Ensure that both inlet air meshes are installed.
- Make sure that inlet air filter is installed.
- Locate the oxygen concentrator so that audio alarms can be heard.

WARNING

Avoid using the oxygen concentrator in the presence of pollutants, smoke or fumes. Do not use the oxygen concentrator in the presence of flammable anesthetics, cleaning agents, or other chemical vapors.

CAUTION

Do not obstruct the air inlet or gas outlet when operating the device. Obstruction of air circulation or proximity to a heat source may lead to heat buildup inside oxygen concentrator, resulting in shutdown or damage.

CAUTION

The PRIZMA O2 oxygen concentrator is designed for continuous use. For optimal sieve bed lifetime, the product should be used regularly.

CAUTION

Do not use the oxygen concentrator without the inlet air meshes and inlet air filter installed. Aspirated particles can damage the oxygen concentrator.

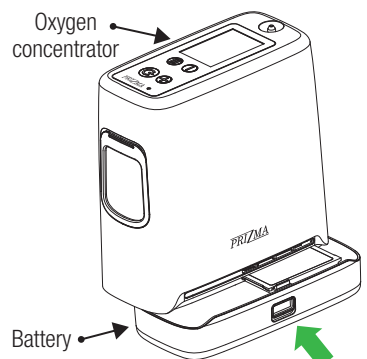
6.2 Battery installation/removal

To install the battery into the oxygen concentrator:

- Put the battery to the bottom of the oxygen concentrator.
- Slide the battery in until it is latched to oxygen concentrator (a “clicking” sound will be heard).


You will hear a single beep and see the display light up briefly before shutting off. This indicates that the concentrator is properly connected to the battery.

NOTE: Before using the device on battery power for the first time, the battery must be fully charged. For details, refer to Chapter 6.9, “Guidelines for use, maintenance and storage of batteries”.

To remove the battery from the oxygen concentrator:

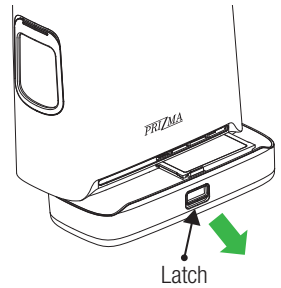- Press down the battery latch using your finger.
- Slide the battery out of the oxygen concentrator.

6.3 Connection of external DC power supply

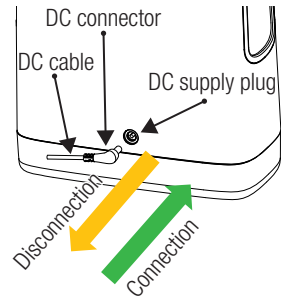
The external DC power supply can be used to charge the battery or to operate the device. If the device operates on an external DC power supply, the battery will not be used and will preserve its capacity. If the external DC power supply is disconnected, the device will automatically switch to battery power.

To connect the external DC power supply to the oxygen concentrator, push the DC connector into the DC supply plug.

If the external DC power supply is an AC/DC adapter, plug the AC connector of the AC/DC adapter into the mains.

After connecting to the external DC power supply, if the oxygen concentrator was turned off, the LCD screen will turn on and indicate that charging is in progress and show the current battery capacity. An audible beep will also be heard.

To disconnect the external DC power supply from the oxygen concentrator, pull the DC connector (do not pull on the DC cable) out of the DC supply plug.

WARNING

Do not disassemble the AC/DC adapter. There are no user-serviceable parts inside. Doing so may lead to failure of the AC/DC adapter and/or pose a safety risk.

WARNING

The AC/DC adapter is not waterproof.

CAUTION

Make sure the AC/DC adapter is placed in a well-ventilated location. Otherwise, it may overheat during operation.

CAUTION

When the AC/DC adapter is disconnected from the mains, also disconnect it from the oxygen concentrator to avoid unnecessary battery discharge.

CAUTION

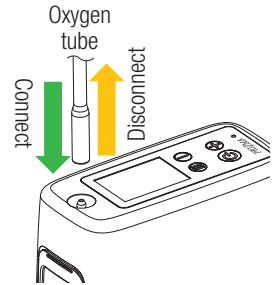
Under certain conditions, the AC/DC adapter may shut down. The green LED may blink or turn off. In this case, disconnect the AC/DC adapter from the mains for at least 10 seconds, then reconnect it to the mains.

6.4 Connection/disconnection of oxygen tube to/from the oxygen concentrator

To connect the oxygen tube, push the end of the oxygen tube onto the connector nipple.

To disconnect the oxygen tube, pull the end of the oxygen tube (do not pull on the tube itself) off the connector nipple.

It is recommended to use the supplied oxygen tube (or another medical-grade oxygen tube of similar length) to ensure proper breath detection and oxygen delivery.

WARNING

To ensure the required oxygen flow, make sure that the nasal cannula is properly connected to the oxygen tube and that the oxygen tube is not kinked or pinched in any way.

WARNING

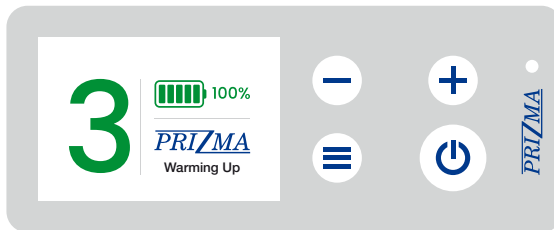
Clean and disinfect the oxygen tube and nasal cannula daily, or replace them with new ones, to prevent infection.

6.5 Turning the oxygen concentrator on


Press and hold On /Off button  until manufacturer's logo appears on the display and the compressor inside the oxygen concentrator starts running.

After a few seconds, the screen will switch to the main screen. The main screen will show the selected oxygen flow setting and the battery capacity.

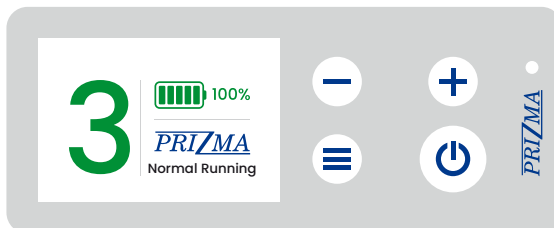A two-minute countdown for warm-up of the oxygen concentrator will start. During this time, the oxygen concentration will increase but may not yet reach the specified value. If the ambient temperature is very low, additional warm-up time may be required.

When the oxygen concentrator is ready, the status on the screen will change to “Normal Running”.

6.6 Setting of oxygen flow rate

Use the  and  buttons to adjust the oxygen flow to the desired level. The current setting is shown on the display.

**WARNING**

The settings of other models or brands of oxygen therapy equipment may not correspond to the settings of this device.

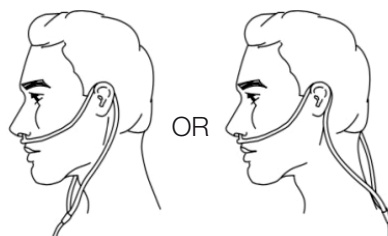
**WARNING**

The settings of this device may not correspond to the settings for devices that provide a continuous-flow oxygen.

6.7 Application of nasal cannula on your face and breathing through nose

Ensure that the nasal cannula is correctly positioned on your face and that you are breathing through your nose. Ensure that the tubing is not excessively tight.

In case of discomfort, skin irritation, or pain around the nose and ears, reposition the cannula. In case of significant irritation or sores, consult your physician.

The oxygen concentrator will detect start of each breath intake and deliver a portion of oxygen at that moment (it is Auto-Mode). If the breathing rate changes, the oxygen concentrator will detect these variations and deliver oxygen only when needed. Sometimes, if you take irregular, quick breaths between regular breaths, the oxygen concentrator may ignore such breaths. This is normal, as the oxygen concentrator senses and follows your regular breathing pattern. A green light will flash each time at the start of an oxygen delivery portion.

Recommended nasal cannula model:

Model: NASAL CANNULA, 1165002

Manufacturer: INTERSURGICAL LTD.

NOTE: Do not use pediatric (low-flow) nasal cannulas.

Use only nasal cannulas meeting the following specifications:

- Suitable for high flow
- Crush-resistant
- Tube/oxygen concentrator connector size: internal diameter 5.0 mm, length 18.4 mm
- Suitable for gas flow rates up to 15 liters per minute (LPM) at a maximum pressure of 0.25 bar
- Suitable for adults

- Compliant with ISO 10993 standards

WARNING

If no breath is detected for 60 seconds, the “No Breath Detected” alarm will be activated. The device will automatically switch to Auto Mode and continue delivering oxygen at a rate of 20 breaths per minute. When breathing is detected again, the device will clear the “No Breath Detected” alarm and switch to “Normal Running” mode.

WARNING

If you begin to feel unwell or experience discomfort while using this device, stop using it IMMEDIATELY.

WARNING

The oxygen concentrator should provide a flow of high-concentration oxygen. If the oxygen concentration drops below specified value for a prolonged period of time, the “Oxygen Low” alarm will appear on the display. If the alarm persists, contact your equipment provider.

WARNING

To prevent cross-contamination and infection, if the oxygen concentrator is used by more than one patient, the nasal cannula and oxygen tube must be replaced between patients.

WARNING

The oxygen delivery settings of the oxygen concentrator should be periodically reassessed for therapy effectiveness.

WARNING

Geriatric or any other patient unable to communicate discomfort may require additional monitoring and/or a distributed alarm system to convey information about discomfort and/or medical urgency to the responsible caregiver in order to avoid harm.

6.8 Turning off the oxygen concentrator

Press and hold the On/Off button  until the main screen switches to the manufacturer’s logo.

The oxygen concentrator will enter a shutdown procedure that may last several tens of seconds. At the end of the shutdown procedure, the compressor inside the oxygen concentrator will stop and display will turn off.

If oxygen concentrator was connected to an external DC power supply and you do not intend to charge battery or do not plan to use oxygen concentrator for an extended period, disconnect the external DC power supply from the oxygen concentrator.

In addition, if the external DC power supply used is an AC/DC adapter, disconnect the AC connector of the AC/DC adapter from the mains.

6.9 Guidelines for use, maintenance and storage of batteries

**CAUTION**

Use only batteries recommended for use in the PRIZMA O2 oxygen concentrator.

**CAUTION**

It is the responsibility of the user to periodically check, recharge, and replace the battery if necessary.

For operation of the oxygen concentrator on battery power:

- The battery must be properly installed in the oxygen concentrator and charged.
- The external DC power supply should be disconnected from the oxygen concentrator.

During operation of the oxygen concentrator the battery will gradually discharge.

When the oxygen concentrator detects that the remaining battery capacity is low, a low-priority visual and audible alarm will be activated. When the battery is almost empty, the priority of the alarm will increase.

If the battery is low, connect an external DC power supply to the oxygen concentrator or turn off the oxygen concentrator and replace the empty battery with a charged one.

The battery will charge when an external DC power supply is connected to the oxygen concentrator. Observe the charging indicator on the display of the oxygen concentrator and monitor the progress of battery charging. Leaving the oxygen concentrator connected to an external DC power supply after the battery is fully charged will not cause any harm.

NOTE: The battery will not charge if voltage of the external DC power supply is lower than 16V.

NOTE: At the beginning of the charging process of a completely discharged battery, charging may stop and resume several times during the first few minutes after the charging process starts.

NOTE: Before using the device with a new battery for the first time, the battery must be fully charged.

To ensure proper performance and long life of the lithium-ion battery:

— **Keep the battery dry.**

Keep liquids away from the battery.

— **Do not use the battery at extreme temperatures.**

To extend the battery lifetime, avoid use at temperatures lower than 5 °C and higher than 35 °C.

— **Perform complete charge/discharge cycles of the battery.**

To maintain maximum battery life, the battery should be fully charged and discharged down to 0% at least once every 90 days.

— **Label the batteries.**

If you use more than one battery, label each battery (1, 2, 3 or A, B, C, etc.) to make battery tracking easier.

— **Store the battery properly.**

- If you do not plan to use the oxygen concentrator for an extended period, remove the battery (regardless of whether it is charged or empty) from the oxygen concentrator.
- Before storage, charge the battery to 40–50% of its nominal capacity.
- Store the battery in a cool, dry place.
- **RECHARGE THE STORED BATTERY TO 40–50% OF ITS NOMINAL CAPACITY AT LEAST ONCE EVERY 90 DAYS.**
- **DO NOT STORE BATTERIES FOR LONGER THAN ONE YEAR WITHOUT RECHARGING.**
- Do not store batteries loosely in a box or drawer where they may be short-circuited by metal objects.

7. Cleaning and Maintenance

The PRIZMA O2 oxygen concentrator requires no special maintenance for continuous and reliable operation.

Parts of the PRIZMA O2 oxygen concentrator that require cleaning or disinfection:

	Parts	Nasal cannula	Housing	Inlet air meshes
Cleaning / Disinfecting method	Alcohol	○		
	Mild dish detergent		○	○
	Water rinse			○

Use a commercially available disinfectant. Follow the instructions provided by the disinfectant manufacturer.

7.1 Oxygen tube and nasal cannula

The oxygen tube and nasal cannula should be cleaned with alcohol as needed and replaced regularly. It is recommended to use the original oxygen tube (or another medical-grade oxygen tube of similar length) and nasal cannula, to ensure proper breath detection and oxygen delivery.

WARNING

To prevent cross-contamination and infection, if the oxygen concentrator is used by more than one patient, the nasal cannula and oxygen tube must be replaced between patients.

7.2 Housing of the oxygen concentrator

The outside of the housing should be cleaned weekly using a cloth dampened with mild liquid dish detergent and water.

CAUTION

Before cleaning, turn off the device, remove the battery and disconnect the device from external DC power supply and the mains.

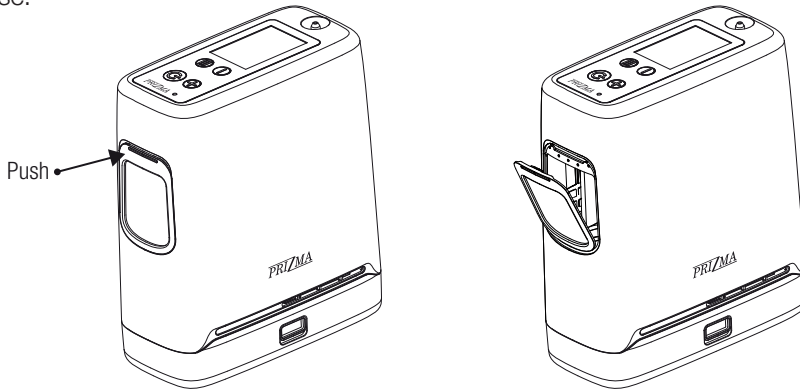
WARNING

Do not use alcohol, isopropyl alcohol, ethylene chloride or petroleum-based cleaners to clean the housing of the oxygen concentrator or the inlet air meshes. Do not hose down or submerge the oxygen concentrator or its accessories in any liquid and do not allow any liquid to enter the device. Doing so may result in electric shock and/or damage.

7.3 Inlet air meshes

To ensure unobstructed airflow, weekly cleaning of the inlet air meshes is recommended. It may be necessary to clean the inlet air meshes more frequently in a dusty environment. Wash the mesh with mild liquid detergent and water, rinse it with water, and dry it before reuse.


To purchase additional inlet air meshes, contact your equipment provider.

7.4 Inlet air filter

The inlet air filter is intended to filter small particles and remove water droplets. It is recommended to replace it after 3500 hours of operation, or when the molecular sieve is replaced.

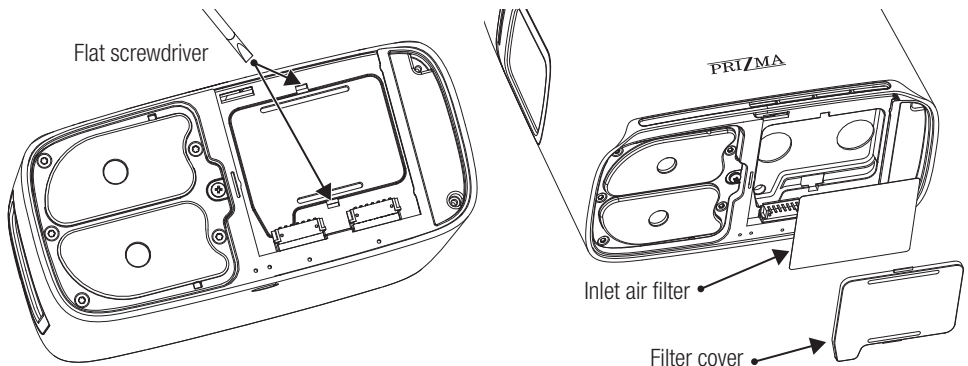
To access the inlet air filter pad, detach the battery from the oxygen concentrator. The inlet air filter pad is located at the bottom of the oxygen concentrator, under the plastic filter cover. Remove the filter cover using a flat screwdriver. Replace the inlet air filter and reassemble the filter cover and battery.

7.5 Output filter

The output filter is intended to protect the user from inhalation of small particles in the oxygen output flow. The output filter is located under the oxygen tube connector. A special tool is required to remove the oxygen tube connector.

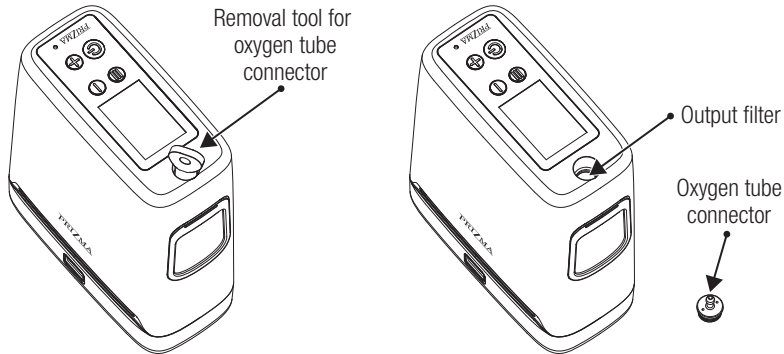A special tool for removing the oxygen tube connector to replace the output filter will be provided with the purchase of additional output filters.

It is recommended to replace the output filter when the molecular sieve module is replaced.

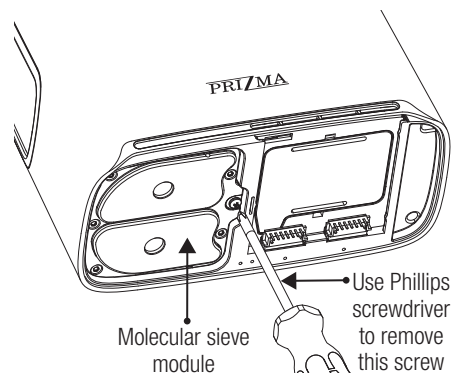
WARNING

To prevent cross-contamination and infection, if the oxygen concentrator is used by more than one patient, the output filter must be replaced between patients.

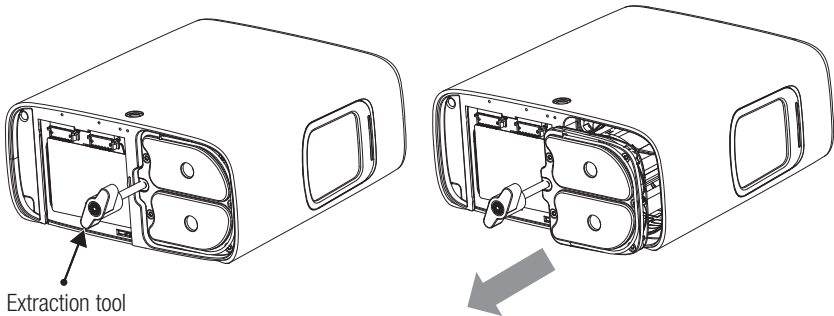
7.6 Replacement of the molecular sieve module

7.6.1 Disassembly of the molecular sieve module

1. Turn off the oxygen concentrator.
2. Disconnect the external DC power supply if it is connected.
3. Remove the battery from the oxygen concentrator.
4. Lay the oxygen concentrator on its front side, so the underside is accessible.
5. Remove the screw securing the molecular sieve module using a Phillips screwdriver.

6. Screw the supplied extraction tool into the threaded hole on the molecular sieve module until the molecular sieve module is released. The extraction tool is provided with the purchase of an additional molecular sieve module.
7. Pull the molecular sieve module out of the oxygen concentrator.
8. Unscrew the supplied extraction tool from the threaded hole on the molecular sieve module and store the extraction tool for future use.

7.6.2 Assembly of the molecular sieve module

1. Remove the dust caps from the new molecular sieve module. Keep the dust cap locations free of dust or debris. The molecular sieve module should be installed as quickly as possible after removal of the dust caps.
2. Insert and push the new molecular sieve module into the oxygen concentrator until it is fully seated.
3. Using a Phillips screwdriver, fasten the bottom of the molecular sieve module with the screw.
4. Connect the external DC power supply to the oxygen concentrator. Turn on the device.
5. Press the function button  once to enter the notifications screen. Press the  button to select Sieve, then press the function button  to confirm the selection. The screen with information about the molecular sieve will be displayed.
6. On this screen, press and hold both the  and  buttons for 5 seconds until the molecular sieve operating time resets to 00000H.
7. Press the  button to return to the home screen.

WARNING

Do not use any molecular sieve module other than the original one. Use of non-original molecular sieves modules may lead to safety hazards, deterioration of equipment performance and void of warranty.

7.7 Other servicing and maintenance activities

**WARNING**

The oxygen concentrator should be inspected at least once a year for correct operation of the device and alarm system.

**WARNING**

Do not disassemble or modify the oxygen concentrator or any accessories or attempt any maintenance other than that described in this instruction manual.

There are no user-serviceable parts inside the oxygen concentrator and external DC power supplies.

Disassembly creates a hazard of electric shock and will void your warranty.

Do not use lubricants on the oxygen concentrator or its accessories, as this will void the warranty.

Do not tamper seal label as this will void the warranty.

For events not listed in this instruction manual, contact your equipment provider for repair by authorized personnel. To avoid damage during transportation, carefully pack the oxygen concentrator and its accessories before shipment (in the original packaging, if possible).

NOTE: The equipment supplier is under no obligation to provide a loaner unit while servicing of a malfunctioning unit is in progress.

8. Quick Troubleshooting

This device is designed in accordance with the product safety standard EN 60601-1, which applies to medical electrical equipment.

The troubleshooting/repair table below is intended to assist in resolving common problems encountered during the use of the oxygen concentrator. If the suggested actions are not effective, switch to an alternative oxygen source and notify the oxygen concentrator supplier for repair.

Please do not attempt any other repairs yourself. Unauthorized repairs will void the warranty.

Symptom	Possible cause	Recommended action
The device does not power on when the On/Off button is pressed.	The On/Off button was not pressed long enough.	Press and hold the On/Off button for at least 2 seconds.
	The battery is discharged or no battery is attached to the oxygen concentrator.	Connect an external DC power supply or replace the empty battery with a charged one.
	The AC/DC adapter is not properly connected to the mains or to the oxygen concentrator.	Check the connection to the mains and to the oxygen concentrator. Verify that the green light on the AC / DC adapter is illuminated.
	The DC cable is not properly connected to the oxygen concentrator.	Check the connection to the oxygen concentrator and to the cigarette lighter plug.
	Malfunction of the oxygen concentrator.	Contact your equipment provider for servicing.
It is not possible to adjust or change the oxygen flow setting.	Malfunction of the oxygen concentrator.	Contact your equipment provider for servicing.
The oxygen concentrator is not detecting breath properly.	The breath sensor requires calibration.	To auto-calibrate the breath sensor: Turn the device off for a few seconds. Then turn the device on, but do not use it for 5 minutes. After 5 minutes, the calibration will be complete.
No oxygen.	The oxygen concentrator is not turned on.	Press and hold the On/Off button for a few seconds to start the oxygen concentrator.
	The nasal cannula is not properly positioned, the oxygen tube is not connected to the oxygen concentrator, or the oxygen tube is kinked or obstructed.	Check the position of the nasal cannula, the oxygen tube connection to the oxygen concentrator, and ensure that the tube is not kinked or obstructed.
Audible or visual alarm on the display.	Please refer to Chapter 9.	Please refer to Chapter 9.
In case of problems not listed, turn oxygen concentrator off and switch to backup oxygen supply. Contact your POC provider immediately.		

9. Explanation of Notifications and Alarms

**WARNING**

If you are unable to hear or see alarms, do not have normal tactile sensitivity, or cannot communicate discomfort, consult your clinician before using this device.

**WARNING**

Audible notifications (65dBA - 70dBA at a distance of 1 meter) are used for user warning purposes. The maximum distance at which audible notifications can be heard depends on the surrounding noise level.

**WARNING**

Avoid noisy environments to ensure that the audible alarms can be noticed.

9.1 Notifications

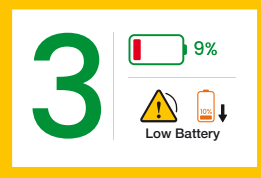
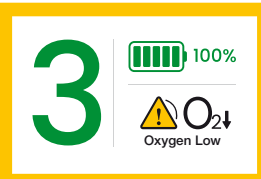
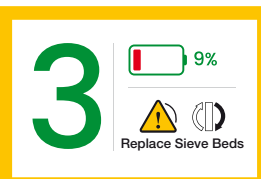
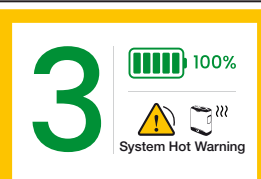
In case of problems during operation of the oxygen concentrator, audible and visual alarms will be generated and the user should take the appropriate actions. When an audible alarm is heard by the patient or the operator, the operator should approach the display on the top side of the device and identify the displayed alarm symbol in order to take the appropriate action.

The alarm threshold values are factory-adjusted and cannot be changed by the user. The electronics inside the PRIZMA O2 monitor various parameters during operation and use an intelligent alarm system to indicate equipment failures. Mathematical algorithms and time delays are used to reduce the probability of false alarms, while still ensuring correct notification of alarm conditions.

In addition, in case of problems, a visual notification will appear on the display of the oxygen concentrator. If more than one alarm condition is detected, the alarm with the highest priority will be displayed. If several alarms of the same priority are present, the most recently occurring alarm will be displayed.

If the alarms indicate a prolonged interruption in the delivery of concentrated oxygen, the user should switch to a backup oxygen source. To silence an audible alarm, press simultaneously the buttons  and . To reactivate an audible alarm, press simultaneously the buttons  and .

9.2 Low priority alarms


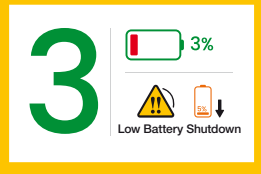
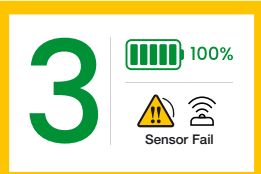
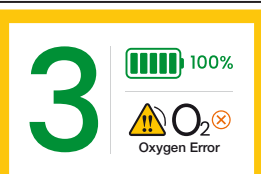
Alarm Message	Description	Action
	<p>Low Battery Battery power is low with less than 10% battery volume.</p>	<p>Connect to external power or turn off the power and replace with a fully charged battery.</p>
	<p>Oxygen Low The oxygen concentrator has been producing oxygen at a slightly low level (<82%) for more than 10 minutes.</p>	<p>If the condition persists, please contact your equipment supplier.</p>
	<p>Service Soon The oxygen concentrator requires servicing at the earliest convenience. The concentrator is operating to specification and may continue to be used.</p>	<p>Contact your equipment supplier to arrange for service.</p>
	<p>Replace Sieve Beds Sieve beds replacement is proposed within 30 days.</p>	<p>Contact your equipment supplier to arrange for service or order new sieve beds from the manufacturer.</p>
	<p>Battery Hot Warning The battery temperature is nearing the temperature limit while the concentrator is running on battery power.</p>	<p>If possible, move the oxygen concentrator to a cooler location or power unit with an external power supply and remove battery. If condition persists, contact your equipment supplier.</p>
	<p>System Hot Warning The oxygen concentrator temperature is nearing temperature limit.</p>	<p>If possible, move the oxygen concentrator to a cooler location. Ensure air intake and outlet vents have clear access and particle filters are clean. If the condition persists, contact your equipment supplier.</p>

Alarm Message	Description	Action
	<p>Warming Up Oxygen concentration is <87% two minutes after the device's start up and at least 10 breaths have been detected within the last minute.</p>	<p>Wait a few minutes to see if the oxygen concentrator improves (alarm will clear). If condition persists, a secondary alarm will sound. Follow the instructions for that alarm or contact your equipment supplier.</p>

9.3 Medium priority alarms

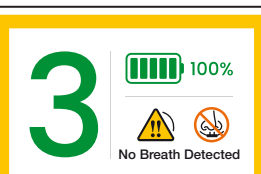Medium priority alarm messages are accompanied by a triple beep, repeating every 25 seconds, and a flashing yellow frame around display.

Alarm Message	Description	Action
	<p>Battery Hot Battery has exceeded temperature limit while concentrator is running on battery power.</p>	<p>Move the oxygen concentrator to a cooler location OR power device with an external power supply and remove battery. If condition persists, contact your equipment supplier. and switch to a backup source of oxygen.</p>
	<p>System Hot The system temperature exceeds the limits temperature, and the device automatically shut down.</p>	<p>Cool down the device and then try again, If the condition persists, Contact your equipment supplier and switch to a backup source of oxygen.</p>
	<p>Fan Failure Fan rotation is not detected. the machine will automatically shut down after 10 seconds.</p>	<p>Contact your equipment supplier and switch to a backup source of oxygen.</p>
	<p>High Pressure The pressure exceeds the Max. limit value, the device automatically shut down after 10 seconds.</p>	<p>Contact your equipment supplier and switch to a backup source of oxygen.</p>

Alarm Message	Description	Action
 <p>3  3%  Low Battery Shutdown</p>	<p>Low Battery Shutdown Battery remaining is less than 5%. Device will automatically shut down soon.</p>	<p>Attach external power supply OR power down and insert a fully charged battery. OR switch to a backup source of oxygen.</p>
 <p>3  100%  System Cold</p>	<p>System Cold The system is cold (<math><2^{\circ}\text{C}</math>). The concentrator will shut down and stop producing oxygen.</p>	<p>Move to a warmer environment to allow the device to warm up before starting it. If condition persists switch to a backup source of oxygen and contact your equipment supplier.</p>
 <p>3  100%  Sensor Fail</p>	<p>Sensor Fail The concentrator's oxygen sensor has malfunctioned.</p>	<p>Contact your equipment supplier and switch to a backup source of oxygen.</p>
 <p>3  100%  Oxygen Delivery Error</p>	<p>Oxygen Delivery Error A breath has been recognized, but proper oxygen delivery has not been detected.</p>	<p>Contact your equipment supplier and switch to a backup source of oxygen.</p>
 <p>3  100%  Oxygen Error</p>	<p>Oxygen Error Oxygen output concentration has been below 50% for 10 minutes.</p>	<p>Contact your equipment supplier and switch to a backup source of oxygen.</p>
 <p>3  100%  No Breath Detected</p>	<p>No Breath Detected The concentrator has not detected a breath for 60 seconds.</p>	<p>The oxygen concentrator has not monitored breathing for 60 seconds. Check that the nasal oxygen tubing is attached to the oxygen concentrator, is not kinked, and is properly placed in the nose.</p>

Alarm Message	Description	Action
	System Malfunction The concentrator has stopped producing oxygen.	Contact your equipment supplier and switch to a backup source of oxygen.

9.4 Informational signals

Alarm Message	Description	Action
	Battery Communication Issue Check battery.	Check the connection of your battery and ensure that it is properly attached and latched to the concentrator. If the battery error persists with same battery stop using the battery and switch to a new battery or remove the battery and operate the concentrator using an external power supply.

10. Environment Protection

This device may not be disposed of with household waste. After use of the device, please return the device to the device retailer for disposal. This device contains electrical and/or electronic components that must be recycled per EU Directive 2012/19/EU - Waste Electrical and Electronic Equipment (WEEE). Non-infectious used accessories can be disposed of as residential waste. The disposal of infectious accessories must be made via an approved waste disposal company. Names and addresses can be obtained from the local municipality.

Lithium-ion batteries should be recycled. Recycle batteries according to national and local regulations. Contact your local representative for assistance. The batteries must be disposed only in a discharged state at the collection center. In case of not fully discharged batteries, provide for a risk against short circuits. Short circuits can be prevented by isolating the terminals with tape.

11. Electromagnetic Compatibility

1. The oxygen concentrator shall be installed and used in accordance with the electromagnetic compatibility information in this chapter.
2. All connecting cables provided by the manufacturer must be used. The power cables used in the device have passed FCC certification.
3. **WARNING:** Use of non-original cables and/or accessories may result in an increase in emissions or a decrease in electromagnetic immunity.
4. In normal use, the oxygen concentrator and its accessories may cause electromagnetic interference with other diagnostic or treatment equipment or may receive interference from them. Maintain an appropriate distance between the oxygen concentrator and other devices and pay close attention to the proper operation of all devices in use.
5. Portable and mobile RF communication devices may affect the operation of the oxygen concentrator. Therefore, it is recommended to keep the portable and mobile RF communication devices away from the oxygen concentrator or switch them off.
6. The device should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the device should be observed to verify normal operation. If the operation is not normal, the device or the other equipment should be moved.
7. The oxygen concentrator is able to deliver oxygen to the patient within the declared performance levels or generate various alarm conditions. If an EM disturbance occurs, the device will not operate under normal conditions and may not achieve its essential performance. Degradation of performance or shutdown of the device may occur.

Guidelines and Manufacturer's Declaration - Electromagnetic Emission

The oxygen concentrator is intended to be used in the electromagnetic environment specified below, and the user's obligation is to use it in specified electromagnetic environment.

Emission test	Compliance	Electromagnetic Environments - Guidelines
RF emission CISPR 11	Group 1	The oxygen concentrator uses RF energy only for its internal function. Therefore, its RF emissions are very low and it is not likely to cause any interference in nearby equipment.
RF emission CISPR 11	Class B	The oxygen concentrator is suitable for use in all environments, including domestic and those directly connected to the public low-voltage power supply grid that supplies buildings used for domestic purposes.
Harmonic emission IEC61000-3-2	Class A	
Voltage fluctuation / flicker emission IEC61000-3-3	Complies	

Guidelines and Manufacturer's Declaration - Electromagnetic Immunity

The oxygen concentrator is intended to be used in the electromagnetic environment specified below, and the user's obligation is to use it in specified electromagnetic environment.

Immunity Test	IEC60601 Test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge IEC61000-4-2	±8 kV contact ±2, 4, 6, 8 and 15 kV air	±8 kV contact ±2, 4, 6, 8 and 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient IEC61000-4-4	±2 kV for power supply line	±2 kV for power supply line	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC61000-4-5	±1 kV line(s) to line(s) ±2 kV line(s) to earth	±1 kV line(s) to line(s) ±2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC61000-4-11	0% U_T for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315°. 0% U_T for 1 cycle 70% U_T for 25/30 cycles. 0% U_T for 200/300 cycles.	0% U_T for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315°. 0% U_T for 1 cycle 70% U_T for 25/30 cycles. 0% U_T for 200/300 cycles.	Mains power quality should be that of a typical commercial or hospital environment. If the user of the [ME EQUIPMENT or ME SYSTEM] requires continued operation during power mains interruptions, it is recommended that the [ME EQUIPMENT or ME SYSTEM] be powered from an uninterrupted power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical hospital or home environment.

U_T is the AC main voltage prior to application of the test waveforms.

Guidelines and Manufacturer's Declaration - Electromagnetic Immunity

The oxygen concentrator is intended to be used in the electromagnetic environment specified below, and the user's obligation is to use it in specified electromagnetic environment.

Immunity Test	IEC60601 Test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC61000-4-6	3 Vrms 150 kHz - 80 MHz 6 Vrms ISM and amateur frequency bands		Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1.2 \sqrt{P}$ 150 kHz - 80 MHz $d = 1.2 \sqrt{P}$ 80 MHz - 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz - 2.5 GHz P - maximum output power of the transmitter in watts (W) according to the transmitter manufacturer's specification. d - recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked  with the following symbol: 
Radiated RF IEC61000-4-3	10 V/m 80 MHz - 2.7 GHz		

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not be appropriate for all situations. Electromagnetic propagation is affected by the absorption and reflection of buildings, objects and the human body.

- a) Field strength from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted accurately theoretically. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be conducted. If the measured field strength in the location in which the concentrator is used exceeds the above listed applicable RF compliance levels, normal operation of the oxygen concentrator should be verified by observation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the device.
- b) Over the frequency range 150 kHz to 80 MHz the field strengths should be less than 3 V/m.

Recommended Separation Distances between Portable and Mobile RF communication equipment and oxygen concentrator

This oxygen concentrator is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The user of the oxygen concentrator can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and this oxygen concentrator as recommended below, according to the maximum output power of the communication equipment.

Rated maximum output power of transmitter (W)	Separation distance depending to the frequency of transmitter (m)		
	150 kHz - 80 MHz $d = 1.2 \sqrt{P}$	80 MHz - 800 MHz $d = 1.2 \sqrt{P}$	800 MHz - 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters with maximum rated output power not listed above, recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the rated maximum output power of the transmitter in watts (W) according to the transmitter's specification.

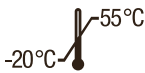
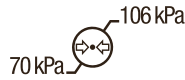
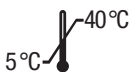
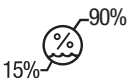
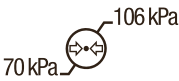
Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2: The guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

12. Specifications

Technical parameters

Model	PRIZMA O2							
Dimensions	Oxygen concentrator without battery: 187 × 88 × 184 mm Oxygen concentrator with battery: 187 × 88 × 217 mm							
Weight	Oxygen concentrator without battery: 1.64 kg Oxygen concentrator with battery: 2.16 kg							
Oxygen output nipple type	EN 13544-2:2002+A1:2009 corrugated nipple							
Battery autonomy (new, fully charged battery)	Flow setting	Base capacity battery [h]		Double capacity battery [h]				
	1	5		10				
	2	4		8				
	3	3		6				
	4	2		4				
	5	1.5		3				
Battery charging time	Up to 2.5 h			Battery model	320120			
	Warm up time							
Approximately 2 minutes to reach 87% oxygen concentration								
Sound level	43 dBA typical at setting 2 Maximum system sound pressure of 61 dBA Maximum system sound power of 66 dBA							
Oxygen concentration	After 2 minutes warming up, 87% - 96% (V/V) at all settings							
Max rate of breathing	40 BPM			Max pressure	0.15 MPa ± 25%			
Inspiratory trigger sensitivity	< 0.12 cmH ₂ O			Flow setting	Levels 1 - 6			
Oxygen delivery	Oxygen volume per breath (mL/breath) and average oxygen flow per minute [mL/min] during normal use for all flow settings (1-6).							
	Flow setting	Breaths per minute						Average oxygen flow [mL/min]
		15	20	25	30	35	40	
	1	14	10.9	8.9	7.4	6.3	5.4	210
	2	29	22	17	14	12	11	420
	3	43	32	26	22	18	16	630
	4	56	43	35	29	25	21	840
	5	69	52	42	36	30	26	1050
6	82	62	50	42	36	31	1260	
± 15% at environment conditions of 101.3 kPa, 20°C, dry air. ± 25% over entire operating range.								

Set points of alarms	<p>≥ 82% – Normal Oxygen symbol (Green)</p> <p>< 82% – Low Oxygen symbol (Yellow) and audible alarm</p>
AC/DC Power Supply	<p>EDACPOWER ELEC., model: EM11012M</p> <p>Input: AC 100 - 240V / 2.0 - 1.0A/ 50 - 60Hz</p> <p>Output: DC 24 V, 5.0 A</p>
DC/DC Power Supply	<p>EDACPOWER ELEC., model: ED1010C</p> <p>DC input: DC 11 - 16V</p> <p>Output: DC 19V, 6.3 A</p>
Protection class against electrical shock	Class II
Transport and storage conditions (Oxygen concentrator & battery)	<p>Temperature: </p>
	<p>Relative humidity:  non condensing</p>
	<p>Pressure: </p>
Operating temperature	
Operating humidity	 non condensing
Operating altitude	Up to +3048m
Operating pressure	<p> Operating outside these range can limit concentrator from meeting stated specifications</p>
Ingress of water or particulate matter into device	<p>IP22</p> <p>The first number 2: Protected against access to hazardous parts with a finger, and the jointed test finger of 12 mm Ø, 80 mm length, shall have adequate clearance from hazardous parts, and protected against solid foreign objects of 12.5 mm Ø and greater.</p> <p>The second number 2: Protected against vertically falling water drops when enclosure is tilted up to 15°. Vertically falling drops shall have no harmful effects when the enclosure is tilted at any angle up to 15° on either side of the vertical.</p>

Expected service life	Oxygen concentrator: 5 years Molecular sieve: 3500 hours Battery: 500 full charge-discharge cycles Output air filter: 3500 hours Inlet air filter: 3500 hours
Shelf lifetime	Oxygen concentrator: Unlimited. Date of manufacture is indicated on the label of the oxygen concentrator.
	Battery: Storage and periodic recharging shall be performed in accordance with the battery storage and maintenance instructions, section 6.9. During storage, the battery shall be recharged at least once every 12 months.
Packaging includes	Oxygen concentrator with battery, AC/DC adapter, Nasal cannula and Instruction manual.

CE 1304

Manufacturer:		PRIZMA KRAGUJEVAC DOO Kumanovska 8, 34000 Kragujevac, SERBIA www.prizma.rs
Production facility		PRIZMA KRAGUJEVAC DOO Dr Zorana Đinđića 13, 34000 Kragujevac, SERBIA
EU representative		PRIZMA GmbH Graf-Adolf-Platz 15 40213 Düsseldorf, GERMANY
Importer		
Version		04/2026

WARRANTY

WARRANTY PERIOD :

Oxygen concentrator main unit: **5 years**

AC adapter with power cord: **24 months**

Battery: **12 months**

Molecular sieve (sieve beds): **12 months** or
3,500 operating hours (whichever occurs first)

Consumables (filters, nasal cannula): **No warranty**

This warranty does not affect mandatory statutory consumer rights.

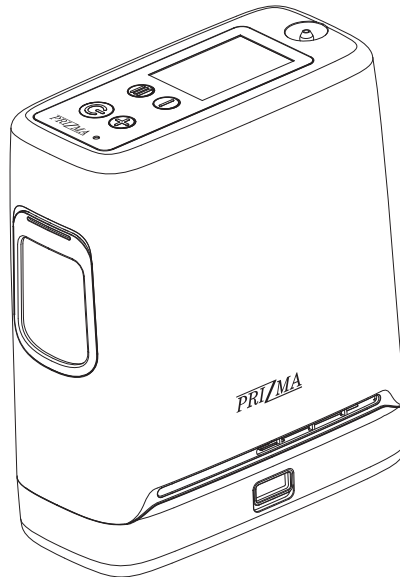
Company PRIZMA KRAGUJEVAC DOO declares:

- The product will work correctly within the warranty period, if you follow attached instructions;
- All manufacturing errors within the warranty period will be solved in a time not longer than 45 days;
- A product that can not be repaired within 45 days, will be at your request replaced with new one;
- All replaced parts for the product in warranty time are guaranteed.

The warranty ends in the following cases:

- If you did not follow the operating instructions
- Repair done by an unauthorized person
- Damage caused by mechanical force of the user or a third party
- Damage caused by natural disaster – flood, fire, lightning... (Force majeure)

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